


Using the CareSens™ Blood Glucose Monitor

SETTING THE CODE

1. Take note of the number in the CODE square on the label of the Test Strip vial, this is the CODE you need to set the meter to.
2. Insert the test strip into either port with black side facing up. Push the strip in gently as far as it will go. The Code number that the meter is set to will appear on the screen for 3 seconds (the code will be 1 if the meter is new). If the blood drop symbol appears before you have changed the Code, just remove the test strip and start again.
3. During the 3 seconds the number is displayed, click the “M” or “C” button to scroll the code numbers up or down **before the blood drop symbol appears on the screen (you have 3 seconds to begin clicking the buttons)**.
4. After selecting the correct code, wait 3 seconds until the blood drop symbol appears. The meter has stored the Code and is now ready for testing.

PERFORMING A BLOOD GLUCOSE TEST

1. Insert the test strip into either port with black side facing up. Push the strip gently until the meter beeps. The Code number that the meter is set to will appear on the screen for 3 seconds. If it is not the same as the Code number on the test strip vial you will need to change it (see above). The flashing drop symbol will then appear - the meter is now ready for you to apply blood to the Test Strip.
2. Using the lancing device, collect a sample of blood which is at least 0.5 microlitres (look on page 29 of the User manual to see the size of the blood drop)
3. While the blood drop symbol is flashing on the screen, apply the blood sample to the white tip at the point of the strip. The blood will be sucked into the white square and the meter will beep.

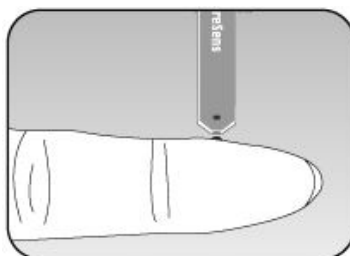

4. The meter will count down from 5 – 1 on the display and the test result, time and date will be shown. This information will automatically be stored in the meters memory.
5. Remove used test strip from the port. The meter will turn off after 3 seconds.

FOR SUPPORT CALL 1800 114 610